

DISPOSABLE LAPAROSCOPIC SPECIMEN RETRIEVAL BAGS





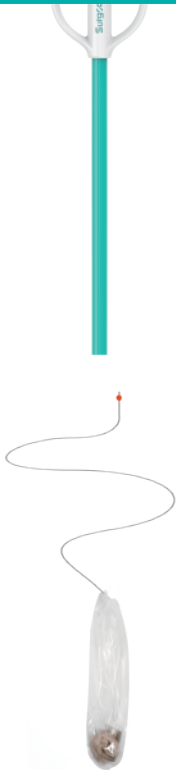
Surgsci
SURGSCI MEDICAL LTD.



PRODUCT INTRODUCTION

- The disposable laparoscopic specimen retrieval bag is mainly used to retrieve human tissue specimens/foreign objects out of the body during minimally invasive surgery.
- The main body is composed of a handle, a shaft and a retrieval bag, which can solve the problem of removal of excisions in minimally invasive surgery.

USE STEPS

INITIAL STATE	EXTENT	TIGHTEN	RELEASE	SEPARATE
				
<p>The initial state of the product when it is taken out of the package</p>	<p>Push the handle to the shaft end to extent the nickel-titanium alloy and open the bag</p>	<p>Put the tissue in the retrieval bag and pull the handle back to tighten the bag</p>	<p>Press the button to unlock the drawstring of the bag</p>	<p>The bag and the drawstring can be separated from the shaft</p>



- Press the release button, the locking mechanism unlocks the drawstring.



- The retrieval bag can be separated from the shaft.

MODEL	DIAMETER	VOLUME (ML)
RB200A	10mm	200
RB400A	10mm	400
RB450A	10mm	600